



RS-LB™ Low Back Conductive Garment

Enable patients to easily take low back treatments at home



Patient puts on the garment just like a belt...



...connects the stimulator, then leans back or lies down and takes a treatment

Helps patients comply

Getting patients to comply with home electrotherapy prescriptions for the back can be difficult. Patients often cannot manage with conventional supplies due to hard-to-reach treatment sites.

RS Medical has solved this dilemma by developing the RS-LB™ Low Back Conductive Garment. It helps patients place the stimulator pads on difficult-to-reach back locations without assistance from others.

The garment makes possible a range of back protocols for treatments using four and eight pads.

Fits most body shapes and sizes

The RS-LB is prescribed for patients using an RS Medical stimulator for back treatments.

The garment utilizes sturdy, flexible and washable fabric and comes in two different sizes along with an extender to fit most body types. Special reusable conductive pads deliver the stimulator's electrical impulse to the body.

Individually set up for each patient

Each patient is individually fit for his or her garment. Electrode pads are arranged for the patient's prescribed treatment. Patients are instructed how to put on the garment for accurate pad placement.

Garment Benefits

- ▣ Helps patients get pads on difficult-to-reach low back areas
- ▣ Supports increased patient compliance
- ▣ Fits most body shapes and sizes

RS-LB Features

❖ Function

- ❖ Enables patients to take treatments at home by applying electrode pads on difficult-to-reach back locations without assistance from others
- ❖ For patients who cannot independently take back stimulation treatments with conventional supplies due to the need to stimulate large body areas, multiple sites and inaccessible locations
- ❖ Assists in correctly positioning pads on back
- ❖ Use with an RS Medical electrical stimulator to transfer electrical impulses from cables through garment snaps and conductive pads to the body

❖ Electrode pads

- ❖ Unique two-sided adhesive electrode pads specifically for garment use
- ❖ Pads can be removed for pad replacement or laundering of garment
- ❖ Re-supply available from RS Medical

❖ Pad pattern

- ❖ Accepts the pad placement protocols for the lower back as defined in the RS Medical Pad Placement Guide for RS Medical 2-Channel and 4-Channel Stimulators

❖ Construction

- ❖ Washable stretch fabric garment
- ❖ Eight adhesive pads provide conductive contact

❖ Size

- ❖ Fits adults with 25-67" waist sizes
- ❖ Small/Med for 25-36"
- ❖ Large for 35-46" plus optional 21" extender to 67" waist

❖ Package contents

- ❖ One RS-LB Lower Back Conductive Garment
- ❖ Eight two-sided adhesive electrode pads
- ❖ Four plastic pad liners
- ❖ One garment cable set
- ❖ Operation Manual
- ❖ Set of snap safety covers

PRESCRIBING INFORMATION

INDICATION: The RS-LB Low Back Conductive Garment and associated accessories are indicated for use with RS Medical Stimulators to facilitate electrode placement and maintenance of electrode positioning for stimulation treatments of the lower back.

GARMENT PRECAUTIONS: Do not connect stimulator cables to Garment without pads in correct place. Each pad must completely cover its metal snap. Do not allow pads to touch each other during stimulation. Do not place or operate this device on the carotid artery region or head. Do not apply to broken skin; if a rash appears, discontinue use and consult your medical professional. Do not use while driving a vehicle or operating machinery. For single patient use only. Keep out of reach of children.

Please review the Operations Manual provided with the prescribed Stimulator

WARNING: FEDERAL LAW RESTRICTS THE SALE OF ELECTRICAL STIMULATORS ON THE ORDER OF A LICENSED DOCTOR OR PHYSICIAN, UNDER THE LAWS OF THE STATE WHERE LICENSED TO PRACTICE.

Benefits