

4-Channel Bipolar Pad Placement







The electrodes are placed directly on the body, or garment, according to your healthcare professional's instruction or product instruction session. The cable pin ends correspond to channels on the RS Medical Stimulator.















- Orange Channel 1
- White Channel 2
- Blue Channel 3
- Yellow Channel 4





Each channel has two cable pin ends

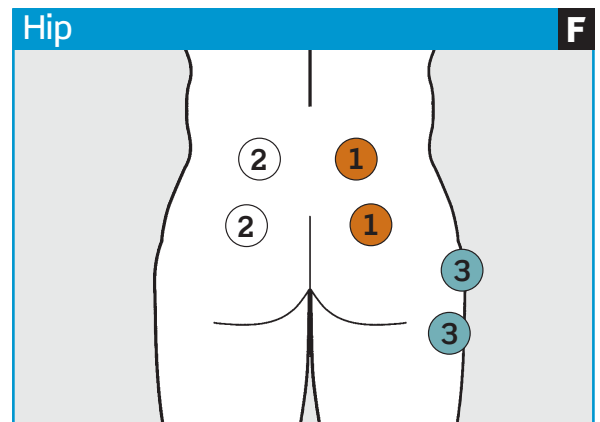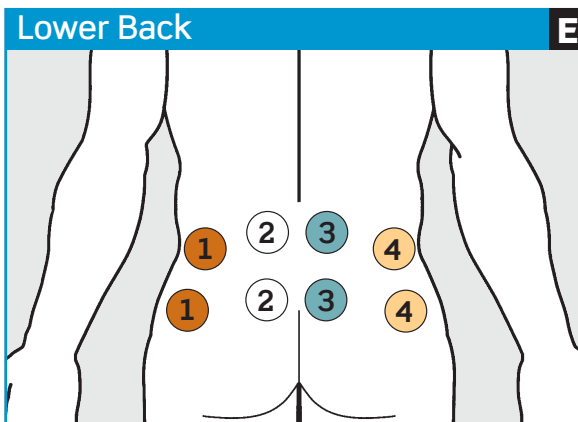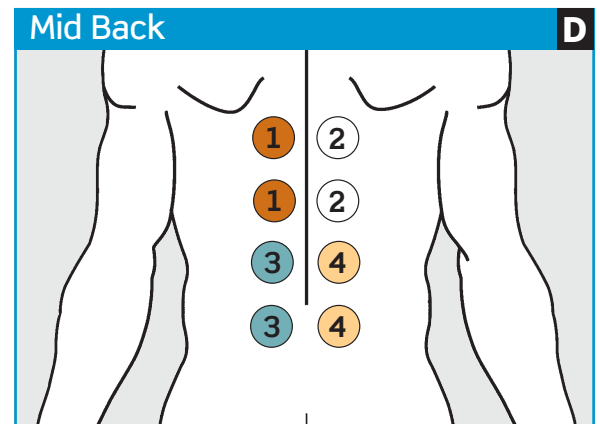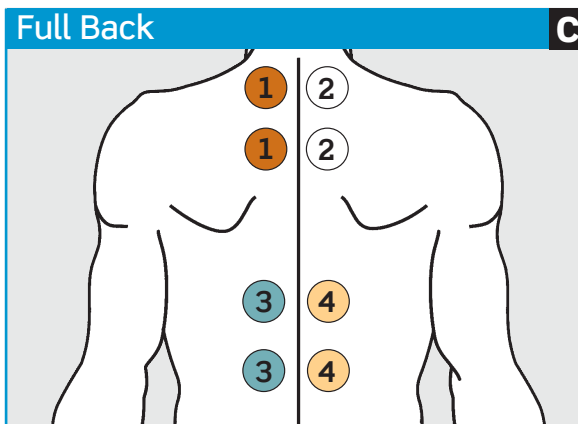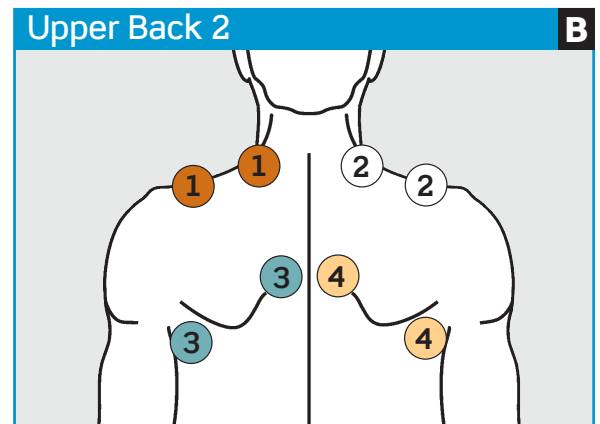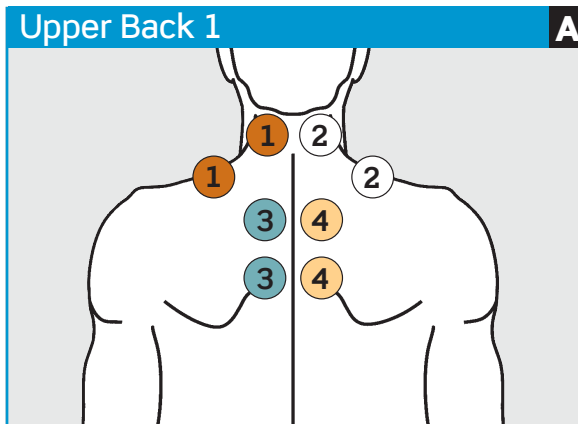

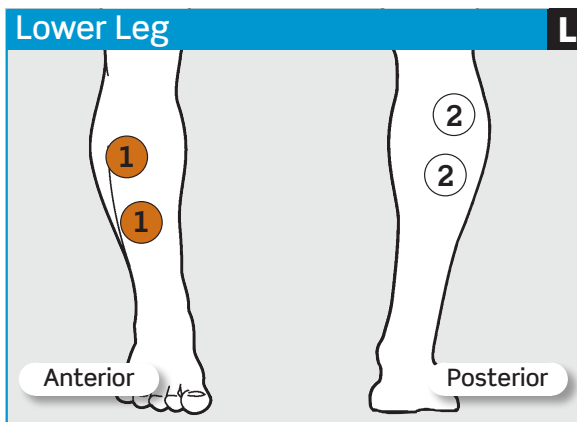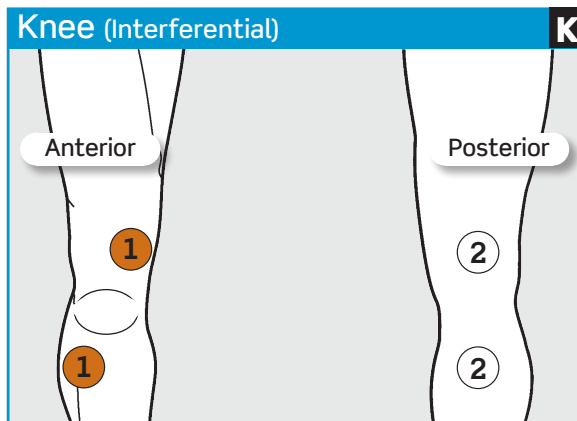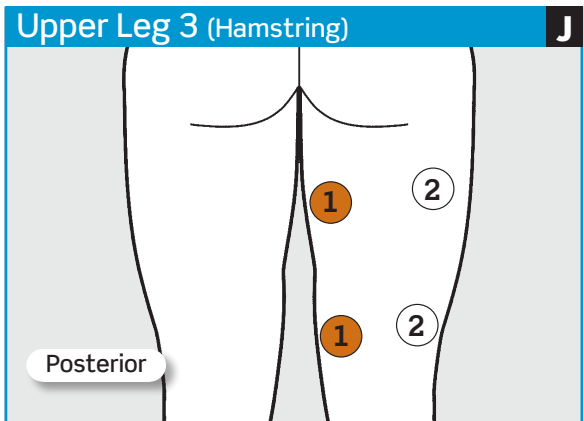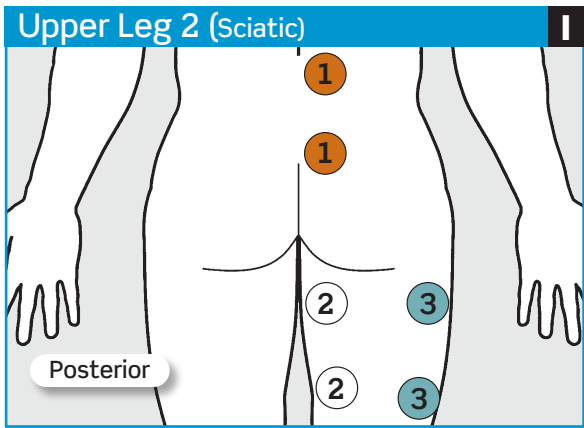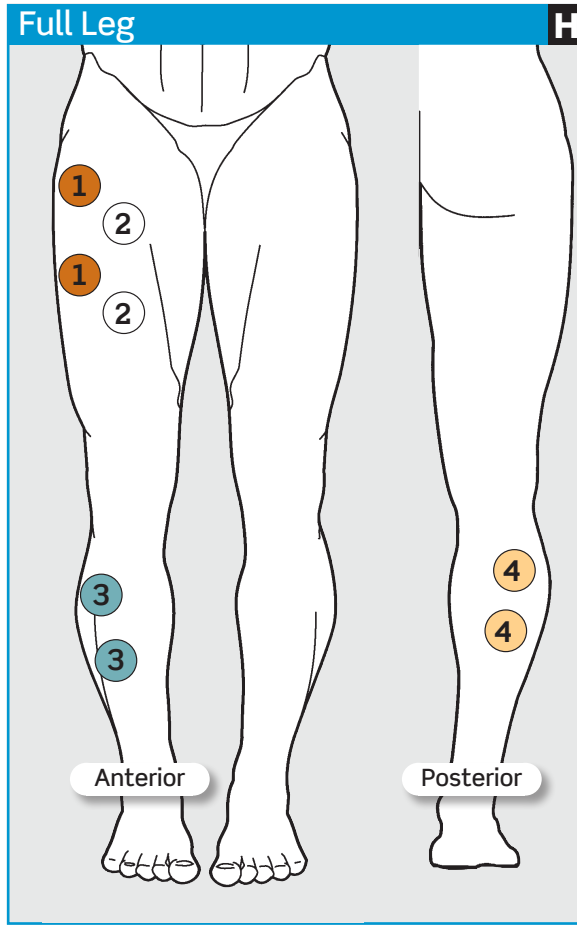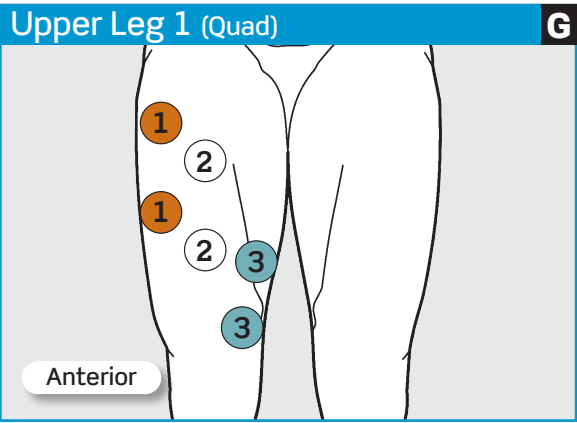

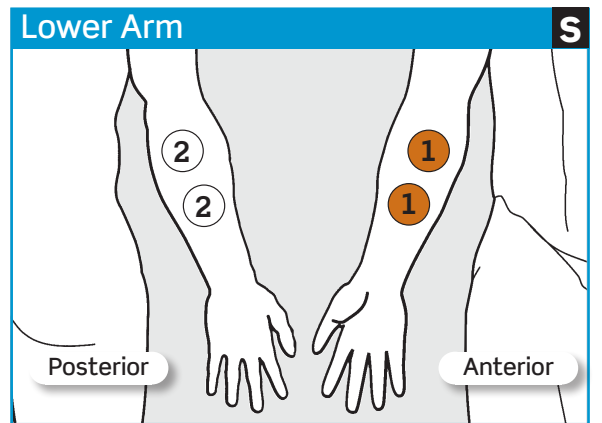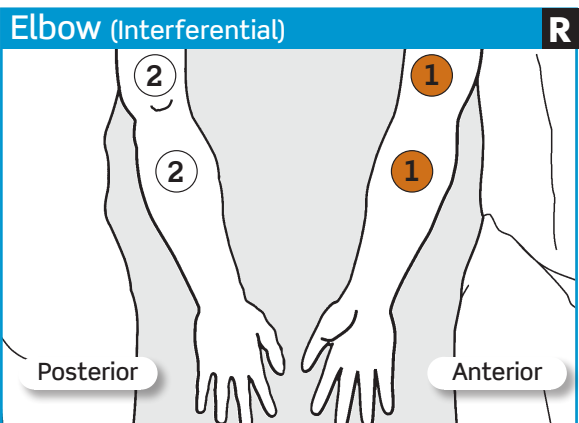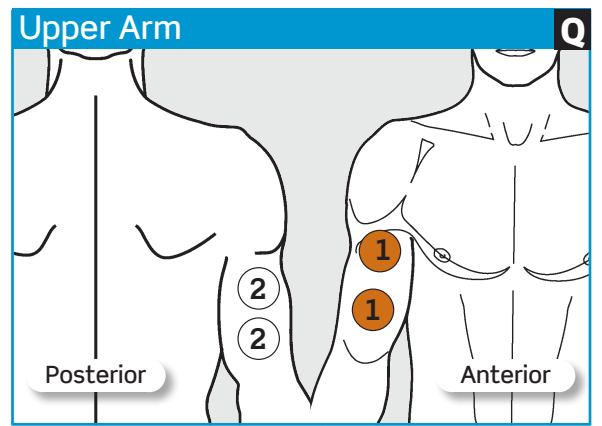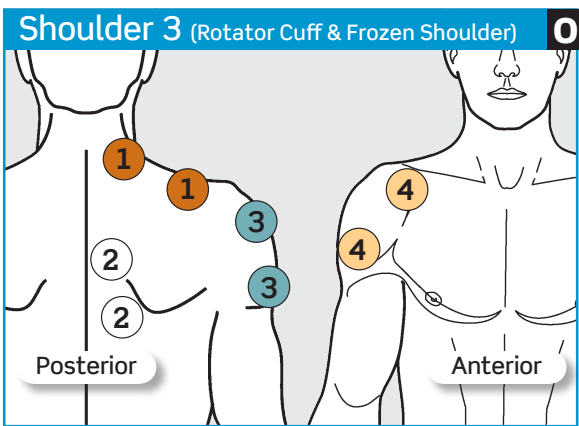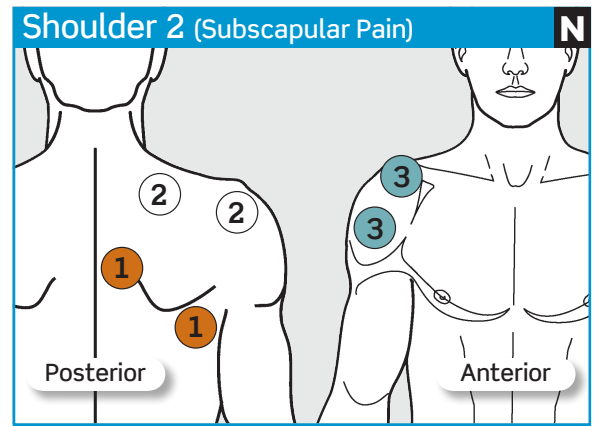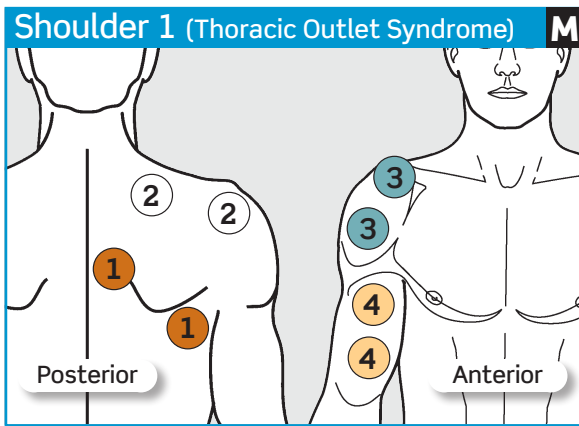

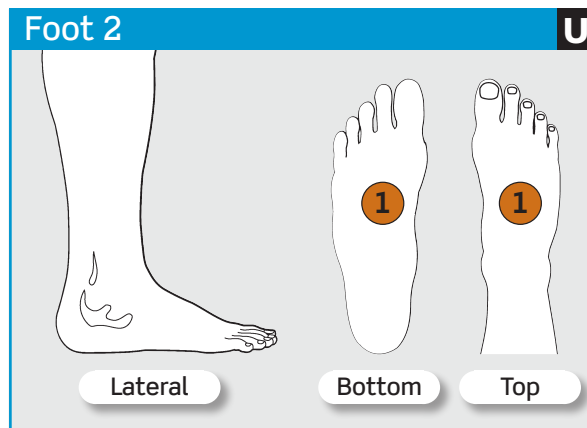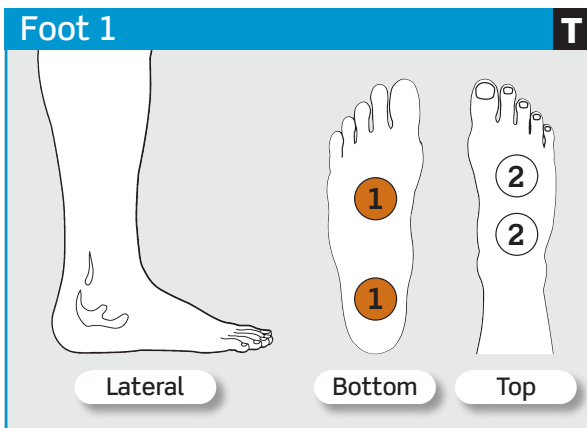

When to order electrodes

Typically, new electrodes are needed every month. The electrodes are disposable and use an adhesive that will dry after prolonged usage or storage. Electrodes should be replaced when they lose their adhesive quality, or when you sense a change in stimulation sensation. If you are concerned about the condition of the electrodes, order fresh ones.

How to order electrodes

Please contact your healthcare professional to order additional electrodes. Otherwise, you may order electrodes directly from RS Medical Customer Service at 800-935-7763.